# Stop H®

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# Stop H®

Rectal gel in single doses  
Hemorrhoidal disorders

## Form and presentation

**Stop H** is a medical device presented in the form of gel in single-dose cannulas of 5 ml.  
The box contains 10 cannulas.

## Properties

Hemorrhoidal disease can cause painful attacks when the hemorrhoidal veins around the anus become inflamed and overly dilated.

**Stop H** is an effective solution to fight against internal and external hemorrhoids.

Thanks to its composition based on active ingredients of natural origin, this medical device helps relieve the symptoms of the hemorrhoidal attack such as: irritation, itching, burning and anal discomfort.

## The active ingredients

**Antiphlogisticum K** has decongestant properties and soothes the first symptoms of inflammation thanks to glycyrrhetic acid.

**Unitamuron H-22 (tamarind seed extract)** is a substance similar to vegetable hyaluronic acid. It forms a protective and soothing film that protects the anal mucous membrane against the aggressions.

**Horse chestnut extract** is known for its decongestant, astringent and toning properties for the local circulation.

The gel is also composed of lubricating substances (glycerin) facilitating defecation.

## Mode and route of administration

Rectal route.

Use 2 single doses per day, one dose in the morning and one in the evening, preferably after evacuation, for 5 days.

1. Remove the cap, which closes the cannula.

2. Gently squeeze the tube to release a bead of gel to lubricate the tip of the cannula.

3. Insert the cannula into the anal orifice, avoiding its full insertion, and apply moderate pressure to empty the contents of the dose on the anal mucosa.

4. Remove the cannula while maintaining gentle pressure, and then discard the single dose after use.

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[www.kenkopara.com](http://www.kenkopara.com)

### Precautions for use:

- Should not be used in persons under 15 years of age.
- Do not ingest.
- Can be used in pregnant or breastfeeding women.
- A medical advice is recommended in the event of a history of anorectal surgery or in the event of hypersensitivity to one of its components.
- In case of bleeding or absence of beneficial effects after 5 days, stop the treatment and consult a doctor.
- This medical device contains natural substances that make it colorful; after applying it, pay particular attention to ensure that your clothes are not stained.

### Possible side effects of Stop H :

Like any active product, this medical device may cause some people to experience more or less troublesome effects, in case of prolonged or repeated use:

- Risk of anal burning sensations,
- Allergic type reactions,
- A possibility of local irritation,

Tell your doctor or pharmacist about any unwanted or bothersome effects not mentioned in this leaflet.

You can also report adverse reactions directly to the laboratories manufacturing this medical device via the email address: [pharmacovigilance@pharmaghreb.com](mailto:pharmacovigilance@pharmaghreb.com)

### Storage conditions

It is recommended to store **Stop H** at room temperature, away from heat and humidity.

Keep out of reach and sight of children.

Do not use after the expiration date Indicated on the box.

Do not dispose of any medical device in wastewater.

### Composition :

Purified Water, sorbitol, vegetable glycerin, Antiphlogysticum K (butylene glycol, panthenol, urea, glycyrrhetic acid, allantoin, sodium benzoate, potassium sorbate), Unitamuron H-22 (tamarind seed extract, pentylene glycol), sodium carboxymethylcellulose, Horse chestnut (maltodextrin, horse chestnut bark extract), benzyl alcohol.

Manufactured by Pharmaghreb Laboratories

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