STOP H®

Suppositoires

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Hemorrhoidal disorders

Form and presentation

Stop H is a medical device presented in the form of suppositories of 1,4 g.

The box contains 10 suppositories.

Properties

Hemorrhoidal disease can cause painful attacks when the hemorrhoidal veins around the anus become inflamed and overly dilated.

Stop H is an effective solution to fight against internal hemorrhoids. Thanks to its composition based on active ingredients of natural origin, this medical device helps to repair the anal epithelium while promoting the regression of inflammation and irritation.

In particular, it helps to improve the symptoms of hemorrhoidal crisis by relieving pain, burning and pruritus during defecation.

The active ingredients

Antiphlogisticum K has decongestant properties and soothes the first symptoms of inflammation thanks to glycyrrhetic acid.

Unitamuron H-22 (tamarind seed extract) is a substance similar to vegetable hyaluronic acid. It forms a protective and soothing film that protects the anal mucous membrane against the aggressions.

Horse chestnut extract is known for its decongestant, astringent and toning properties for the local circulation.

Mode and route of administration

Rectal route.

Use 2 suppositories per day, one in the morning and one in the evening, preferably after stools evacuation, during 5 days.

Hold the suppository from its pointed end in order to insert it from its flat end.

Precautions for use:

- Should not be used in persons under 15 years of age.
- Can be used in pregnant or breastfeeding women.

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BP7-1080 CEDEX TUNIS

www.kenkopara.com

- Do not ingest.
- A medical advice is recommended in case of hypersensitivity or allergy to any of the components.
- In case of bleeding or absence of beneficial effects after 5 days, stop the treatment and consult a doctor.
- This medical device contains natural substances that make it colorful; after applying it, pay particular attention to ensure that your clothes are not stained.

Possible side effects of Stop H :

Like any active product, this medical device may cause some people to experience more or less troublesome effects, in case of prolonged or repeated use:

- Risk of anal burning sensations,
- Allergic type reactions,
- Risk of development of congestive rectitis,

Tell your doctor or pharmacist about any unwanted or bothersome effects not mentioned in this leaflet.

You can also report adverse reactions directly to the laboratories manufacturing this medical device via the email address: pharmacovigilance@pharmaghreb.com

Storage conditions

It is recommended to store **Stop H** in its original packaging, at a temperature below 25°C.

Keep out of reach and sight of children.

Do not use after the expiration date Indicated on the box.

Do not dispose of any medical device in wastewater.

Composition :

Semi-synthetic glycerides, talc, Antiphlogysticum K (butylene glycol, panthenol, urea, glycyrrhetic acid, allantoin, sodium benzoate, potassium sorbate), Unitamuron H-22 (tamarind seed extract, pentylene glycol), silica, Horse chestnut (maltodextrin, horse chestnut bark extract).

Manufactured by Pharmaghreb Laboratories

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