# pivalex®

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## 1) What is pivalex and in which case to use it?

### Forms and presentations :

pivalex gingival gel, tube of 15 g  
pivalex gingival solution, bottle of 20 ml with an applicator brush.

### Properties :

Due to its composition based on rhubarb extract and salicylic acid, pivalex soothes mouth ulcers and inflammations of the mouth and sensitive and weakened gums. pivalex contributes to appease the gums and limit dental plaque, therefore favoring purified breath. pivalex is also used to prevent gums redness, swelling and bleeding.

## 2) what are the informations that should be known before use pivalex ?

### Never use pivalex :

If you are allergic to any of the ingredients. If in doubt, it is essential to seek the advice of your doctor or pharmacist.

### Warnings and precautions for use :

Do not swallow.

Does not replace brushing your teeth after each meal.

### Use in children :

This medical device should not be used in children under 12 years old.

### Pregnancy and breastfeeding :

It is preferable not to use this medical device during pregnancy and breastfeeding.

In general, during pregnancy or breastfeeding, always seek the advice of your doctor or pharmacist before using a medical device.

## 3) How to use pivalex?

### Route of use :

Local route.

### Mode of use :

#### pivalex gingival gel, tube of 15 g :

Place a small amount of gel directly on the gum (or on the clean finger) and apply by massaging sensitive areas with the clean finger.

#### pivalex gingival solution, bottle of 20 ml with an applicator brush :

Apply the solution on the lesions using the applicator brush.

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[www.kenkopara.com](http://www.kenkopara.com)

#### Recommended dose :

It is recommended to use 2 to 4 times a day.

#### Recommendations for use :

- pivalex is a bucco-dental care product.
  - Do not rinse.
  - For more effectiveness, it is best to apply the gel or solution after brushing your teeth.
  - the simultaneous or successive use of other products for buccogingival application is not recommended.
- Contact your doctor or pharmacist if in doubt.

#### 4) What are the possible side effects of pivalex?

Same as any active product, this medical device can cause more or less bothersome effects in some people :

- Allergic reactions including rashes and itching.
- A slight tingling may be felt due to the presence of alcohol.
- Yellow discoloration of the teeth or oral mucosa, reversible when the care is stopped.

Tell your doctor or pharmacist about any unwanted or bothersome effects not mentioned in this leaflet.

#### 5) How to store pivalex ?

It is recommended to store pivalex at room temperature in a dry place, away from heat.

Keep out of the reach and sight of children.

Do not use after the expiry date stated on the box, tube or bottle.

Do not throw any medical device into wastewater.

#### 6) Additional informations

##### Composition :

##### Composition of pivalex gingival gel, applicator tube of 15 g :

Purified water, 96% V/V ethanol, glycerin, triethanolamine, rhubarb extract, salicylic acid, carbomer, sodium saccharin.

##### Composition of pivalex gingival solution, bottle of 20 ml with an applicator brush :

96% V/V ethanol, purified water, sodium hydroxide, rhubarb extract, salicylic acid.

Manufactured by Pharmaghreb laboratories

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